# KVISTBERGA™

---

## PERFOLIGHT

Art.

V5, V10, V11, V15, V20  
E1, E2

Bruksanvisning  
SVENSKA

User manual  
ENGLISH

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## PERFOLIGHT

Foldable and one-piece ramps. Art no. V5, V10, V11, V15, V20, E1, E2

- Check that no damage has occurred to the ramps during transport. The ramps should be easy to fold out and back. (not models V5, V10 and E1)
- Read carefully through the instructions and warnings before you use the ramps. Keep the instructions for future use.
- Each ramp has a sign that states the maximum load the ramp may be used for. **The Maximum load is for each ramp.** That is, a pair of ramps can take twice the indicated weight, assuming that the load is evenly distributed. **The maximum load per ramp is shown with this symbol:**

- The sign also states the maximum level difference the ramps are suitable for. **Take care and use the ramps with good judgement.** The recommended maximum height is shown with this symbol:

- Place the ramps on the ground with the transport lock facing upwards. Fold out the ramp to its full length (V11, V15, V20, E2).
- Place the ramp with the flap resting firmly on the upper surface. The underside of the flap has an anti-slip mat to prevent the ramps from slipping on the surface. However, take care when in use so that no part of the wheelchair catches on the edge and pulls the ramp along with it. The distance between the ramps should be adjusted so that each of the wheelchair's wheels passes along the centre of its corresponding ramp's width.
- When transporting, Fold the ramp in half. Always carry the ramp by both its handles, V5, V10 and E1 only has one handle.
- If the ramp are damaged in any way, they must not be used before they have been repaired. Please contact your local dealer or Kvistberga for more information and spare parts.

### Warranty/Guarantee

The guarantee is for 2 years. Damages to the ramp due to misuse or handling not described as earlier are not covered by this guarantee.

### Care and reconditioning

The ramps should be kept free from dirt and snow so that the grip of the load-bearing surfaces is retained. The ramps can be cleaned with all normal types of cleaning agent. A high-pressure jet washer is recommended.

### Disassembling and recycling

All included materials are 100% recyclable. Remove all plastic parts and deposit them for recycling of hard plastics. Aluminium parts should be deposited as scrap metal.

### Intended usage

Ramps are medical devices intended to be used only to assist transportation of disabled persons, seated in wheelchairs or scooters. Ramps enable the passage of elevated points (f.e. Stairs, curbs, car loading edges) with or without the assistance of a helper. Ramps thereby enable patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited.

### WARNING!

Be careful during folding/unfolding the ramp. Fingers or other body parts of a user or/and assistant may be trapped between device's elements. Pay attention to all gaps between parts to avoid injury.

Kvistberga Group AB  
Lancashirevägen 5C  
SE-730 60 RAMNÄS Sweden

phone: +46-(0)21-66 100  
fax +46-(0)21-66 233  
email: [info@kvistberga.se](mailto:info@kvistberga.se)

**KVISTBERGA™**

---