KVISTBERGA™

PERFOLIGHT

Art.

U1, U2, U2/3, U3

Bruksanvisning
SVENSKA

User Manual
ENGLISH

PERFOLIGHT Teleskopiska modeller - U1, U2, U2/3, U3

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
PERFOLIGHT Telescopic ramps U1, U2, U2/3, U3

- Check that no damage has occurred to the ramps during transport. The ramps should be easy to fold out and back again or extend and retract.
- Read carefully through the instructions and warnings before you use the ramps. Keep the instructions for future use.
- Each ramp has a sign that states the maximum load the ramp may be used for. **The Maximum load is for each ramp.** That is, a pair of ramps can take twice the indicated weight, assuming that the load is evenly distributed. **The maximum load per ramp is shown with this symbol:**

- The sign also states the maximum level difference the ramps are suitable for. **Take care and use the ramps with good judgement.**

- **The recommended maximum height is shown with this symbol:**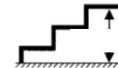

- Place the ramp with the lower end resting on the ground. Release the transport lock. Then extend the telescopic part to the desired length. Place the ramps with the flap resting firmly on the upper surface. The underside of the flaps has an anti-slip mat to prevent the ramps from slipping on the surface. Take care when in use so that no part of the wheelchair catches on the edge and pulls the ramp along with it. The distance between the ramps should be adjusted so that each of the wheelchair's wheels passes along the centre of its corresponding ramp's width.
- When transporting, retract the ramps with care and close the transport lock. Always carry the ramp by its handle.
- If the ramp is damaged in any way, it must not be used before it has been repaired. Please contact your local dealer or Kvistberga for more information and spare parts.

Warranty/Guarantee

The warranty is 2 years. Damages to the ramp due to misuse are not covered by this guarantee.

Care and reconditioning

The ramps should be kept free from dirt and snow so that the grip of the load-bearing surfaces is retained. The ramps can be cleaned with all normal types of cleaning agent. A high-pressure jet washer is recommended.

Disassembling and recycling

All included materials are 100% recyclable. Remove all plastic parts and deposit them for recycling of hard plastics. Aluminium parts should be deposited as scrap metal.

Intended usage

Ramps are medical devices intended to be used only to assist transportation of disabled persons, seated in wheelchairs or scooters. Ramps enable the passage of elevated points (f.e. Stairs, curbs, car loading edges) with or without the assistance of a helper. Ramps thereby enable patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited.

WARNING!

Be careful during folding/unfolding the ramp. Fingers or other body parts of a user or/and assistant may be trapped between device's elements. Pay attention to all gaps between parts to avoid injury.

Kvistberga Group AB
Lancashirevägen 5C
SE-730 60 RAMNÄS Sweden

phone: +46-(0)21-66 100
fax +46-(0)21-66 233
email: info@kvistberga.se

KVISTBERGA™